




Homecare Suppliers

Ramp Selection Guidelines

888-252-2205

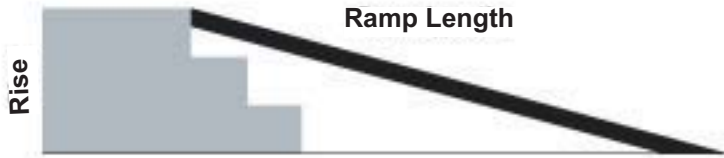

- To select your ramp, first determine the rise for your application.
- Find your max rise and find the proper ramp in the left column.
- Never exceed a slope greater than 2" on 12" with an occupied chair or scooter.
- Never exceed a slope greater than 3" on 12" with an unoccupied chair or scooter.

Ramp Model Number	Ramp Description	Max Rise Occupied (2:12 slope ratio)	Max Rise Unoccupied (3:12 slope ratio)
SFW230	Singlefold 2' x 30"	4"	6"
SFW330	Singlefold 3' x 30"	6"	9"
SFW430	Singlefold 4' x 30"	8"	12"
SFW530	Singlefold 5' x 30"	10"	15"
WCR530	Multifold 5' x 30"	10"	15"
WCR630	Multifold 6' x 30"	12"	18"
WCR730	Multifold 7' x 30"	14"	21"
WCR830	Multifold 8' x 30"	16"	24"
WC1030	Multifold 10' x 30"	20"	30"
WC1230	Multifold 12' x 30"	24"	36"
UTW630	Utility Ramp 6' x 30"	12"	18"
UTW730	Utility Ramp 7' x 30"	14"	21"
UTW830	Utility Ramp 8' x 30"	16"	24"
UTW1030	Utility Ramp 10' x 30"	20"	30"
THR832	Threshold 8" x 32"	1/2"	2"
TH1032	Threshold 10" x 32"	3/4"	2.5"
TH1232	Threshold 12" x 32"	1"	3"
TH1632	Threshold 16" x 32"	2"	4"
TH2432	Threshold 24" x 32"	3"	6"
SL330	Solid 3' x 30" Ramp	6"	9"
SL336	Solid 3' x 36" Ramp	6"	9"
SL430	Solid 4' x 30" Ramp	8"	12"
SL436	Solid 4' x 36" Ramp	8"	12"
SL530	Solid 5' x 30" Ramp	10"	15"
SL536	Solid 5' x 36" Ramp	10"	15"
BAR536	Bariatric Multifold 5' x 36"	10"	15"
BAR636	Bariatric Multifold 6' x 36"	12"	18"
BAR736	Bariatric Multifold 7' x 36"	14"	21"
BAR836	Bariatric Multifold 8' x 36"	16"	24"
BPR348	Bariatric Panel 3' x 48" (3 panel)	6"	9"
BPR448	Bariatric Panel 4' x 48" (3 panel)	8"	12"
BPR548	Bariatric Panel 5' x 48" (3 panel)	10"	15"
BPR648	Bariatric Panel 6' x 48" (3 panel)	12"	18"